## Directions for use

# Ambu® Silicone Face Masks

Silicone masks with inflated cuff

<b>English</b>	
Directions for use.....	3
<b>Deutsch</b>	
Bedienungsanleitung.....	9
<b>Dansk</b>	
Brugsanvisning.....	15
<b>Ελληνικά</b>	
Οδηγίες χρήσης.....	21
<b>Español</b>	
Manual de instrucciones.....	27
<b>Suomi</b>	
Käyttöohje.....	33
<b>Français</b>	
Mode d'emploi.....	39
<b>Italiano</b>	
Manuale d'uso.....	45
<b>Nederlands</b>	
Gebruiksaanwijzing.....	51
<b>Norsk</b>	
Brukerveiledning.....	57
<b>Português</b>	
Manual de instruções.....	63
<b>Svenska</b>	
Instruktionshandbok.....	69

Ambu® are registered trademarks of Ambu A/S, Denmark.  
Ambu A/S is certified according to ISO 9001 and ISO 13485.

## Directions for use

# Ambu® Silicone Face Masks

## Silicone masks with inflated cuff

### 1. Warnings/precautions

#### Caution:

- U.S. federal law restricts this device to sale by or on the order of a physician (USA only).
- For use by CPR trained personnel only. The proper application of a face mask to obtain a tight seal should be trained in particular. Make sure that such personnel are made familiar with the contents of this manual.

### 2. Description

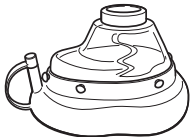
Ambu® **Silicone Face Masks** sizes 0A, 0 (with boring), 0, 2, 3/4, 5 and 6 with inflated cushion are designed for use with manual resuscitators, automatic resuscitators and ventilators.

#### Size 0A with a 15mm ISO connector

The mask is transparent. The cuff of the 0A model is filled with foam and the inflation is not adjustable.


#### Sizes 0, 2, 3/4, 5 and 6 with a 22 mm ISO connector

The mask domes are transparent. The soft cuff is inflatable and the cuff inflation may be adjusted.

### Size 0 with boring

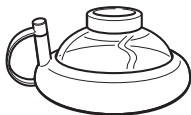The model 0 with boring is designed for use with the Ambu® **Mark III and the Ambu® Mark IV resuscitator** on small children down to three years of age/ approx. 15 kg of body weight. The leakage from the boring is designed to limit the ventilation pressure to approx. 4 kPa (40 cm H<sub>2</sub>O) **and shall not be used for any other application than that.**


## 3. Operating instructions

### Ventilation with a face mask

Apply the mask firmly to the patient's face to achieve a tight seal. Hold the mask tight against the face while maintaining the correct head tilt to keep the airway open.

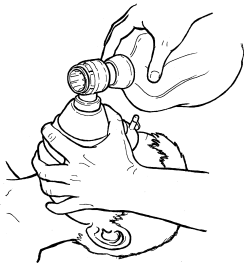**It is very important to thoroughly train correct application of the face mask before any attempt is made to use a resuscitator.**

## 4. Cleaning - disinfecting - sterilising

<ul style="list-style-type: none"> <li>● Applicable</li> <li>○ Not applicable</li> </ul>		Methods						
		Cleaning		Disinfecting - sterilising				
		Washing		Disinfecting			Auto-claving	
Mask		Manual washing	Washing machine (WM)	W.M. heat disinfecting	Boiling	Chemical	121° C	138° C
After each use	Silicone face masks sizes 0, 2, 3/4 & 5	●	●	●	●	●	●	●
	Silicone face mask with foam filled cuff size 0A	●	●	●	○	●	○	○

The table shows applicable methods for cleaning, disinfecting and sterilising.

### Detergents and chemical disinfectants

Numerous brands of detergents and chemical disinfectants are available. Use only brands that are compatible with the mask materials to avoid reduction in the lifetime of the materials. See list of materials on page 6. Follow the instructions of the manufacturer of the detergent or chemical disinfectant for dilution and exposure time. Substances containing phenol should be avoided.

Information on selected detergents and chemical disinfectants verified for compatibility with the mask materials is available on request from the local Ambu representative or Ambu A/S.

**Note:** If residuals of detergents or disinfectants are allowed to dry on the mask materials, surfaces may become impaired and the product lifetime may become reduced.

### The cleaning, disinfecting and sterilising process

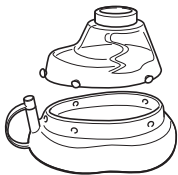
The following steps are generally recommended. Select proper methods for the mask parts in question according to the table above.

## 4.1. Disassembly of the mask (recommended)


Unbutton or remove mask cuff from dome.

### Caution!

Before autoclaving remove stopper from the inflation tube.

*Disassembled mask*

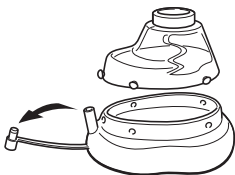*Disassembled mask with stopper removed from the inflation tube.*

## 4.2. Cleaning of parts

### Manual washing or automatic washing machine

Wash the parts in warm water using a detergent suitable for the mask materials cf. paragraph 5. Rinse all parts thoroughly in clean water to remove all detergent residues.

If surface cleaning and/or disinfecting of the outer surfaces of the mask are carried out, make sure that the detergent and/or disinfectant is compatible with the materials of the mask and be sure to remove the detergent/disinfectant by rinsing with water. If residues are allowed to dry on the mask, the lifetime of the materials may be reduced.

An automatic washing machine with a programme designed for washing anaesthesia breathing equipment may be used.

### 4.3. Disinfecting and/or sterilising

Select heated or chemical disinfection according to local standards for disinfection and the table of applicable methods.

#### Washing machine - Heat disinfecting programme

Automatic washing machines designed for medical equipment will normally include program cycles for heat disinfection.

#### Boiling

Use clean water, heat and boil parts for e.g. 10 minutes to disinfect.

#### Chemical disinfecting

Follow the instructions of the manufacturer of the chemical disinfectant for dilution and exposure time. After exposure of the mask parts to the chemical disinfectant, rinse thoroughly in clean water to remove all residues.

#### Autoclaving

Use standard autoclaving equipment adjusted for max. 138°C.

#### Caution!

Before autoclaving remove stopper from the inflation tube.

### 4.4. Drying and cooling

Leave the parts to dry and/or cool completely before reassembling the mask.

### 4.5. Inspection of parts

After cleaning, disinfecting and/or sterilising carefully inspect all parts for damage or excessive wear and replace, if necessary. Some methods may cause discolouration of rubber parts without impact on their lifetime. In case of materials deterioration e.g. cracking, the parts should be replaced.

### 4.6. Assembling

Reassemble the mask.

## 5. Specifications

### Weight

Size 0A:	app. 28 g.
Size 0 with boring:	app. 39 g.
Size 0:	app. 38 g.
Size 2:	app. 63 g.

Size 3/4:	app. 78 g.
Size 5:	app. 80 g.
Size 6:	app. 105 g.

Operating temperature range:

-40°C to 50°C (-40°F to 122°F)

Storage temperature range:

-40°C to 70°C (-40°F to 158°F)

### Parts

Mask cuff size 0A:
Mask dome:
Mask cuff:
Mask connector:

### Materials

Silicone rubber with foam filling
Polysulphone
Silicone rubber
Silicone rubber

### Parts list

Cat. No.	Size	Connector
Blue		
000 313 000	Face mask, size 0A	15 mm ISO connector
000 314 000	Face mask, size 0 with boring	22 mm ISO connector
000 315 000	Face mask, size 0	22 mm ISO connector
000 316 000	Face mask, size 2	22 mm ISO connector
000 312 000	Face mask, size 3/4	22 mm ISO connector
000 317 000	Face mask, size 5	22 mm ISO connector
000 319 000	Face mask, size 6	22 mm ISO connector

### Note:

These directions for use may be updated without further notice.  
Copies of the current version are available from the manufacturer.

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Ambu B.V.